SPECIFICATIONS ROMED CLEANSING WIPES REF: ABW



Identification

Product name: ROMED cleansing wipes

Ref: ABW

Materials

Fabric Non-woven (thermbond) impregnated

Dimensions $200 \pm 10 \text{ mm } \text{ x } 110 \text{ mm} \pm 10 \text{ mm}$ Active ingredient 80% ethanol, 5% isopropanol

Inactive ingredient Water pH 6.5 – 7.5

Purpose

For the cleaning of small surfaces (~0.5 m²).

Mode of application

Take out a single tissue and wipe the objects to be cleaned. No additional wiping necessary.

Caution

- The tissues are pre-moisturized with an alcoholic liquid which is easily inflammable.
- Ensure that the container is always closed tightly after use.
- Be careful with surfaces sensitive to alcohol (e.g. Plexiglas).
- Avoid contact with eyes. If contacted, flush eyes with copious quantities of water.
- If taken in, DO NOT induce vomiting, but obtain medical attention immediately.
- Stop use if irritation and redness develop on the skin. Ask a doctor if conditions persist for more than 72 hours.

If the container is always closed tightly after each use, and stored under the recommended storage conditions it can be expected that the product will remain in good condition for 2-3 months after the first use. The alcohol will evaporate while the container is opened or not closed properly, and will evaporate faster at higher ambient temperatures.

Never use wipe twice
Flammable, keep away from fire or flame
For external use only
Keep out of reach of children
Do not use after expiry date
Single use, discard after use
Keep at room temperature

Packing

110 wipes in a dispenser, 6 dispensers in an inner box, 6 inner boxes in an export carton